

D-Check C5D

HEMATOLOGY CONTROLS

INTENDED USE

D-Check C5D is a control designed to monitor values on Coulter hematology analyzers. Please refer to the assay table for specific instrument models.

SUMMARY AND PRINCIPLE

It is an established laboratory practice to use a stable control to monitor the performance of diagnostic tests. This control is composed of stable materials that provide a means of monitoring the performance of hematology blood cell counters. It is sampled in the same manner as a patient specimen.

REAGENTS

D-Check C5D is an *in vitro* diagnostic reagent composed of human erythrocytes, simulated leukocytes, and mammalian platelets suspended in a plasma-like fluid with preservatives.



PRECAUTION

D-Check C5D is intended for *in vitro* diagnostic use only by trained personnel.



WARNING:

POTENTIAL BIOHAZARDOUS MATERIAL

For *in vitro* diagnostic use. Each human donor/unit used in the preparation of this product has been tested by a FDA licensed method/test and found to be negative or non-reactive for the presence of HBsAg, Anti-HCV, NAT testing for HIV-1, HCV (RNA) and HIV-1/2. Each unit is also negative by a serological test for Syphilis (RPR or STS). Because no test method can offer complete assurance that infectious agents are absent, this material should be handled as potentially infectious. When handling or disposing of vials follow precautions for patient specimens as specified in the OSHA Bloodborne Pathogen Rule (29 CFR Part 1910, 1030) or other equivalent biosafety procedures.



STABILITY AND STORAGE

Store D-Check C5D upright at 2 - 8°C (35- 46°F) when not in use. **Protect tubes from overheating and freezing.** Unopened tubes are stable through the expiration date. Opened tubes are stable for 14 days or 14 thermal cycles (uses), whichever comes first, when handled properly.

INDICATIONS OF DETERIORATION

After mixing, product should be similar in appearance to fresh whole blood. In unmixed tubes, the supernatant may appear cloudy and reddish: this is normal and does not indicate deterioration. Other discoloration, very dark red supernatant or unacceptable results may indicate deterioration. **Do not use the product if deterioration is suspected.**



INSTRUCTIONS FOR USE

1. Remove tubes from the refrigerator and allow them to warm to room temperature (15 to 30°C or 59 to 86°F) for 15 minutes before mixing.
2. To mix, hold a tube horizontally between the palms of the hands. **Do not pre-mix on a mechanical mixer.**
 - a) Roll the tube back and forth for 20 – 30 seconds: occasionally invert the tube. Mix vigorously, but do not shake.
 - b) Continue to mix in this manner until the red cells are completely suspended. Tubes stored for a long time may require extra mixing.
 - c) Gently invert the tube 8 – 10 times immediately before sampling.
3. Prime instrument with whole blood before control analysis.
4. Analyze the sample as instructed in the Quality Control section of the Operator's Manual for your instrument.
5. After sampling:
 - a) If tube has been open for sampling, clean residual material from the cap and tube rim with a lint-free tissue. Replace the cap tightly.
 - b) Return tubes to refrigerator within 30 minutes of use.

QC FILE SETUP PROCEDURE FOR BECKMAN COULTER DxH 800

2 DIMENSIONAL BARCODE SETUP:

1. At the workstation, select **Menu, Set Up,** and then **Quality Control.** Select **New Control From Bar Code** tab. A dialog box appears that reads "Waiting for 2D bar code to be scanned from assay sheet..."
2. Scan the 2D barcode using the hand help scanner. The scanned lot numbers will appear. Click on **Select All** and **OK.**

QC FILE SETUP PROCEDURE FOR BECKMAN COULTER LH700 Series, LH500, GENS AND HMX:

2 DIMENSIONAL BARCODE SETUP: Upload CBC/Diff Assay and Expected Range Values

1. At the Command Center, select **System Set Up.**
2. Select the **Quality Assurance** icon. Select **Controls** tab and click on **New Control Folder** icon.
3. Select the following: Source – **BCI** or **Coulter**; Type – **5C Series**; Level – **All levels.**
4. Select **Setup New Lot** icon. "Load 5C Series Cell Control Reference Values" dialog box appears. Using the hand help scanner, scan the appropriate 2D barcode. Click on **Close** icon.
5. The assay information will be transmitted to the QC CBC/Diff files.

EXPECTED RESULTS

Verify that the lot number on the tube matches the lot number on the table of assay values. Assay values are determined on well-maintained, properly calibrated instruments using the instrument manufacturer's recommended reagents. Reagent differences, maintenance, operating technique, and calibration may contribute to inter-laboratory variation.

PERFORMANCE CHARACTERISTICS

Assigned values are presented as a Mean and Range. The Mean is derived from replicate testing on instruments operated and maintained according to the manufacturer's instructions. The Range is an estimate of variation between laboratories and also takes into account inherent imprecision of the method and expected biological variability of the control material.

Assay values on a new lot of control should be confirmed before the new lot is put into routine use. Test the new lot when the instrument is in good working order and quality control results on the old lot are acceptable. The laboratory's recovered mean should be within the assay range.

For greater control sensitivity each laboratory should establish its own mean and acceptable range and periodically reevaluate the mean. The laboratory range may include values outside of the assay range. The user may establish assay values not listed on the Assay Sheet, if the control is suitable for the method.

LIMITATIONS

The performance of this product is assured only if it is properly stored and used as described in this insert. Incomplete mixing of a tube prior to use invalidates both the sample withdrawn and any remaining material in the tube.

TECHNICAL ASSISTANCE AND CUSTOMER SERVICE

For technical assistance or additional information, please call your dealer or local distributor. If there is no, you may call DIAGON Ltd. Technical Service at +36 1-369-6500.



DIAGON LTD.

Baross str.48-52.

Budapest

Hungary-1047

Tel: +36 1 369-6500

Web: www.diagon.com