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CLINICAL CHEMISTRY MULTICALIBRATOR

REF CAL1010 10x3 mL

In Vitro Diagnostic Use

INTENDED USE

CHEMISTRY MULTICALIBRATOR is a human lyophilized calibration serum joined to biological additives and bacteriostatic agents. The values indicated are useful to calibrate clinical chemistry test in manual and automated analytical procedures. The components concentration are lot-specific.

The exact value and ranges are indicated in the value table.

REAGENT COMPOSITION

Freeze-dried, pooled human sera

REAGENT PREPARATION AND STABILITY

Reconstitute with 3.0 mL of bi-distilled water at 15-25°C.

Replace the stopper and leave to stand for 30 minutes at room temperature out of bright light.

Dissolve mixing slowly, avoiding foam forming, ensure that all lyophilized material is dissolved.

Improper handling or storage, inaccurate reconstitution or microbiological pollution can compromise the results causing wrong results.

The lyophilized control serum is stable until the expiration date indicated on the label if stored tightly closed at 2-8°C, protected from light and contamination.

Reconstituted serum is stable:

1 day a 15-25°C; 7 days a 2-8°C e 1 month a -20°C.

NOTES

- Alkaline Phosphatase values in the reconstituted serum tend to rise during the stability period.
- It is recommended that the reconstituted serum should be allowed to stand for 1 hour at 25°C before measurement.
- The Bilirubin stability, if stored protected from light, is 4 days at 2-8°C.

PRECAUTIONS

The products do not contain dangerous substances or mixtures, according to the EC Regulation n° 1272/2008 or their concentrations are such as not to be considered persistent, bioaccumulative or toxic (PBT). The product is classified and labeled in accordance with EC directives or respective national laws. Sodium azide, less than 0.1%. However, in compliance with the normal prudential rules that everyone has to have managing any chemical or laboratory reagent, in case of contact of reagents with the operator, you must apply the following first aid:

S26 (P305 – P351 – P338): In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

S28 (P302 – P352): After contact with skin, wash immediately with plenty of water.

S36/37/39 (P280): Wear suitable protective clothing, gloves and eye/face protection.

S46 (P301 – P310): If swallowed, seek medical advice immediately and show container or label. If victim is conscious and alert, wash out mouth with water.

S56 (P273): Dispose of this material and its container at hazardous or special waste collection point.

S63 (P304 – P340): In case of accident by inhalation: remove casualty to fresh air and keep at rest. If breathing is difficult, give oxygen and get medical aid.

The human sera present in the mixture were tested with CE-approved and FDA-approved methods and found to be negative for HBsAg, HIV 1 & 2, HCV.

However, since there are no methods to ensure the complete absence of such infectious agents, this product must be handled as if it were risky and potentially capable of transmitting infectious diseases of any kind, in accordance with Good Laboratory Practice standards.

FOR MORE INFORMATION, REQUEST SAFETY DATA SHEET FOR REAGENT (MSDS) AT THE MANUFACTURER.

SIMBOLOGY

CE	CE Mark (EC Directive 98/79)		
	In Vitro Diagnostic		Temperature Limitation
	Consult instructions for use		Contains sufficient for <n> test
	Catalog Number		Use By
	Batch Code		Manufacturer